



## Microsoft Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the ISM nStream and VaultStream platform. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the nStream G3, nCareRX/MX, EasyLink Router, Thin Client, Status Board Engine, MedPresence Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

**Product:** nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the Status Board Engine.

**Date:** May 16, 2019

**Critical Security Bulletins (Obtained from Microsoft):**

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

**Approved Updates:**

January, February, March, April, and May 2019 Updates		
KB Number	Platform	Title
<b>nStream v5.4 Updates</b>		
KB4480970	nStream	2019-01 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB4481480	nStream	2019-01 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7
KB890830	nStream	Windows Malicious Software Removal Tool - January 2019
KB4486563	nStream	2019-02 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB4486563	nStream	2019-02 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4487078	nStream	2019-02 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7
KB4487078	nStream	2019-02 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB890830	nStream	Windows Malicious Software Removal Tool - February 2019
KB890830	nStream	Windows Malicious Software Removal Tool x64 - February 2019
KB4474419	nStream	2019-03 Security Update for Windows Server 2008 R2 for x64-based Systems
KB4489878	nStream	2019-03 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4490628	nStream	2019-03 Servicing Stack Update for Windows Server 2008 R2 for x64-based Systems
KB890830	nStream	Windows Malicious Software Removal Tool x64 - March 2019
KB4493472	nStream	2019-04 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB890830	nStream	Windows Malicious Software Removal Tool - April 2019

KB4499406	nStream	2019-05 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7
KB890830	nStream	Windows Malicious Software Removal Tool - May 2019
KB4499164	nStream	2019-05 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
<b>VaultStream v5.3, 8.0.x, v9.0.x Updates – WS2008</b>		
KB4480970	VaultStream 2008	2019-01 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4481480	VaultStream 2008	2019-01 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4483187	VaultStream 2008	2018-12 Cumulative Security Update for Internet Explorer 11 for Windows Server 2008 R2 for x64-based systems
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - January 2019
KB4486563	VaultStream 2008	2019-02 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4487078	VaultStream 2008	2019-02 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - February 2019
KB4474419	VaultStream 2008	2019-03 Security Update for Windows Server 2008 R2 for x64-based Systems
KB4489878	VaultStream 2008	2019-03 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4490628	VaultStream 2008	2019-03 Servicing Stack Update for Windows Server 2008 R2 for x64-based Systems
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - March 2019
KB4493472	VaultStream 2008	2019-04 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - April 2019
KB4499164	VaultStream 2008	2019-05 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4499406	VaultStream 2008	2019-05 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - May 2019
<b>VaultStream v10.x Updates – WS2012</b>		
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - January 2019
KB4481484	VaultStream 2012	2019-01 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 8.1 and Server 2012 R2 for x64
KB4480963	VaultStream 2012	2019-01 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4480979	VaultStream 2012	2019-01 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB4487038	VaultStream 2012	2019-02 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - February 2019
KB4487000	VaultStream 2012	2019-02 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4487080	VaultStream 2012	2019-02 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 8.1 and Server 2012 R2 for x64
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - March 2019
KB4489907	VaultStream 2012	2019-03 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB4489881	VaultStream 2012	2019-03 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4487258	VaultStream 2012	2019-02 Preview of Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 8.1 and Server 2012 R2 for x64

KB4487016	VaultStream 2012	2019-02 Preview of Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4493446	VaultStream 2012	2019-04 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - April 2019
KB4493478	VaultStream 2012	2019-04 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - May 2019
KB4499408	VaultStream 2012	2019-05 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4497932	VaultStream 2012	2019-05 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
KB4499151	VaultStream 2012	2019-05 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
<b>VaultStream v10.x Updates – WS2016</b>		
KB4480961	VaultStream 2016	2019-01 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4091664	VaultStream 2016	2018-10 Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - January 2019
KB4483229	VaultStream 2016	2018-12 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - February 2019
KB4485447	VaultStream 2016	2019-02 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4487026	VaultStream 2016	2019-02 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4480977	VaultStream 2016	2019-01 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4487006	VaultStream 2016	2019-02 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - March 2019
KB4489882	VaultStream 2016	2019-03 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4489889	VaultStream 2016	2019-03 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - April 2019
KB4493470	VaultStream 2016	2019-04 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4493473	VaultStream 2016	2019-04 Cumulative Update for Windows Server 2016 for x64-based Systems
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - May 2019
KB4498947	VaultStream 2016	2019-05 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4494440	VaultStream 2016	2019-05 Cumulative Update for Windows Server 2016 for x64-based Systems
<b>EasyLink Router v8.0 Updates</b>		
KB4481275	ELR v8.0	2019-01 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4480077	ELR v8.0	2019-01 Security Only Update for .NET Framework 4 on WES09 and POSReady 2009
KB4480087	ELR v8.0	2019-01 Security Only Update for .NET Framework 2.0 SP2 on WES09 and POSReady 2009
KB4483495	ELR v8.0	2019-02 Security Only Update for .NET Framework 3.0 SP2 on WES09 and POSReady 2009

KB4483475	ELR v8.0	2019-02 Security Only Update for .NET Framework 4.0 on WES09 and POSReady 2009
KB4483485	ELR v8.0	2019-02 Security Only Update for .NET Framework 2.0 SP2 on WES09 and POSReady 2009
KB4486463	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4487086	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4486464	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4486465	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4486924	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4487396	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4487385	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4487085	ELR v8.0	2019-02 Security Update for WES09 and POSReady 2009 for x86-based Systems
KB4489493	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4490228	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4489973	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4490501	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4490500	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4490385	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4486468	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4486538	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4486536	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4493341	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4489977	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4489974	ELR v8.0	2019-03 Security Update for POSReady 2009 for x86-based Systems
KB4487989	ELR v8.0	2019-03 Update for POSReady 2009 for x86-based Systems
KB4491443	ELR v8.0	2019-04 Security Update for POSReady 2009 for x86-based Systems
KB4500331	ELR v8.0	2019-05 Security Update for WES09 and POSReady 2009 for x86-based Systems
<b>nCareRX v8.0 and v8.1, nCareMX/nStreamGX v8.0, v9.0, and Status Board Engine, 8.0 and v8.1 Updates</b>		
KB4480970	nCareRX/MX-SBE	2019-01 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems pciclearstalecache_d184a72109a380c13f3e70c14abacd0a01b3f5fd.exe
KB4480960	nCareRX/MX-SBE	2019-01 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4480055	nCareRX/MX-SBE	2019-01 Security and Quality Rollup for .NET Framework 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4480059	nCareRX/MX-SBE	2019-01 Security and Quality Rollup for .NET Framework 4.5.2 for Windows 7 and Server 2008 R2 for x64
KB4480063	nCareRX/MX-SBE	2019-01 Security and Quality Rollup for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
KB4480072	nCareRX/MX-SBE	2019-01 Security Only Update for .NET Framework 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4480076	nCareRX/MX-SBE	2019-01 Security Only Update for .NET Framework 4.5.2 for Windows 7 and Server 2008 R2 for x64
KB4480085	nCareRX/MX-SBE	2019-01 Security Only Update for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64

KB4481480	nCareRX/MX-SBE	2019-01 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
		windows6.1-kb4480063-x64_fa96098bc81237fb45ad887706094c2e0c7a73eb.msu
KB4481481	nCareRX/MX-SBE	2019-01 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4480085-x64_db30a8058583260feee8e81059e005935cc8cb12.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4487345	nCareRX/MX-SBE	2019-01 Update for Windows Embedded Standard 7 for x64-based Systems
KB4487078	nCareRX/MX-SBE	2019-02 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4483458-x64_2e534c379d1ba72d5b2ea5771abd4fbd0ad3e98.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4487121	nCareRX/MX-SBE	2019-02 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
		windows6.1-kb4483483-x64_4b5b9d903814bdeedec8be27aba68e950f1f8a30b.msu
KB4483455	nCareRX/MX-SBE	2019-02 Security and Quality Rollup for .NET Framework 4.5.2 for Windows 7 and Server 2008 R2 for x64
KB4483470	nCareRX/MX-SBE	2019-02 Security Only Update for .NET Framework 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4483483	nCareRX/MX-SBE	2019-02 Security Only Update for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
KB4483474	nCareRX/MX-SBE	2019-02 Security Only Update for .NET Framework 4.5.2 for Windows 7 and Server 2008 R2 for x64
KB4483458	nCareRX/MX-SBE	2019-02 Security and Quality Rollup for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
KB4483451	nCareRX/MX-SBE	2019-02 Security and Quality Rollup for .NET Framework 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4486563	nCareRX/MX-SBE	2019-02 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
		pciclearstalecache_18e1ed872ee0c02212987e995c501c11561e124b.exe
		windows6.1-kb4486563-x64_d327324815ed5bfbabb5ffb860b80b9f59b25a70.msu
KB4486564	nCareRX/MX-SBE	2019-02 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4490511	nCareRX/MX-SBE	2019-02 Update for Windows Embedded Standard 7 for x64-based Systems
KB4491113	nCareRX/MX-SBE	2019-02 Update for Windows Embedded Standard 7 for x64-based Systems
KB4487256	nCareRX/MX-SBE	2019-02 Preview of Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4489885	nCareRX/MX-SBE	2019-03 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4489878	nCareRX/MX-SBE	2019-03 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
		pciclearstalecache_282bf401806e84b3cd6012b8e0417553ac8428fb.exe
		windows6.1-kb4489878-x64_ad02ba6500855333621fecf1b95cc15a8950f2e8.msu

KB4474419	nCareRX/MX-SBE	2019-03 Security Update for Windows Embedded Standard 7 for x64-based Systems
KB4490628	nCareRX/MX-SBE	2019-03 Servicing Stack Update for Windows Embedded Standard 7 for x64-based Systems
KB4489486	nCareRX/MX-SBE	2019-03 Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
		windows6.1-kb4488662-x64_6c483c4681671fe41682a4a6a8750cc6cb170a54.msu
KB4488666	nCareRX/MX-SBE	2019-03 Update for .NET Framework 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4488669	nCareRX/MX-SBE	2019-03 Update for .NET Framework 4.5.2 for Windows 7 and Server 2008 R2 for x64
KB4488662	nCareRX/MX-SBE	2019-03 Update for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
KB4493448	nCareRX/MX-SBE	2019-04 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4493472	nCareRX/MX-SBE	2019-04 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4496880	nCareRX/MX-SBE	2019-04 Update for Windows Embedded Standard 7 for x64-based Systems
KB4498961	nCareRX/MX-SBE	2019-05 Security Only Update for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
		windows6.1-kb4495612-x64_61245f7a7dd9a8e7939f31eb2be1e288713764b8.msu
KB4499406	nCareRX/MX-SBE	2019-05 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4495606-x64_689f245559ffc28ba33ad3675287eff1a014174b.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4495626	nCareRX/MX-SBE	2019-05 Security and Quality Rollup for .NET Framework 4.8 for Windows 7 and Server 2008 R2 for x64
KB4495587	nCareRX/MX-SBE	2019-05 Security Only Update for .NET Framework 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4495612	nCareRX/MX-SBE	2019-05 Security Only Update for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
KB4495588	nCareRX/MX-SBE	2019-05 Security and Quality Rollup for .NET Framework 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2 for Windows 7 and Server 2008 R2 for x64
KB4495606	nCareRX/MX-SBE	2019-05 Security and Quality Rollup for .NET Framework 3.5.1 for Windows 7 and Server 2008 R2 for x64
KB4495627	nCareRX/MX-SBE	2019-05 Security Only Update for .NET Framework 4.8 for Windows 7 and Server 2008 R2 for x64
KB4495596	nCareRX/MX-SBE	2019-05 Security and Quality Rollup for .NET Framework 4.5.2 for Windows 7 and Server 2008 R2 for x64
KB4495593	nCareRX/MX-SBE	2019-05 Security Only Update for .NET Framework 4.5.2 for Windows 7 and Server 2008 R2 for x64
KB4499175	nCareRX/MX-SBE	2019-05 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
		windows6.1-kb4499175-x64_3704acff45ddf163d8049683d5a3b75e49b58cb.msu
		pcclearstalecache_d243a607b50db10ed50f03cff570498018c61a59.exe
KB4499164	nCareRX/MX-SBE	2019-05 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems

<b>HP Thin Client v8.0 and EasyLink Router v8.1 Updates</b>			
KB4480970	ThinClient v8.1	v8.0-ELR	2019-01 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems pclearstalecache_053b8d4fd748b8ae27e33b50541ca8cf3dff1f75.exe
KB4480960	ThinClient v8.1	v8.0-ELR	2019-01 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4487345	ThinClient v8.1	v8.0-ELR	2019-01 Update for Windows Embedded Standard 7 for x86-based Systems
KB4486563	ThinClient v8.1	v8.0-ELR	2019-02 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems windows6.1-kb4486563-x86_acd5206432777e0b9e3596b847fdc92037b24497.msu pclearstalecache_5f77d9baaa9c3de6d3c3e9e375132eb9356342ee.exe
KB4486564	ThinClient v8.1	v8.0-ELR	2019-02 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4490511	ThinClient v8.1	v8.0-ELR	2019-02 Update for Windows Embedded Standard 7 for x86-based Systems
KB4491113	ThinClient v8.1	v8.0-ELR	2019-02 Update for Windows Embedded Standard 7 for x86-based Systems
KB4489885	ThinClient v8.1	v8.0-ELR	2019-03 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4489878	ThinClient v8.1	v8.0-ELR	2019-03 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems pclearstalecache_2cce8d8f493b829de9059e06d65907abdd21018.exe windows6.1-kb4489878-x86_4850c313f7149cbebe5cd718d4efa0e426aebc81.msu
KB4474419	ThinClient v8.1	v8.0-ELR	2019-03 Security Update for Windows Embedded Standard 7 for x86-based Systems
KB4490628	ThinClient v8.1	v8.0-ELR	2019-03 Servicing Stack Update for Windows Embedded Standard 7 for x86-based Systems
KB4493472	ThinClient v8.1	v8.0-ELR	2019-04 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
KB4493448	ThinClient v8.1	v8.0-ELR	2019-04 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4496880	ThinClient v8.1	v8.0-ELR	2019-04 Update for Windows Embedded Standard 7 for x86-based Systems
KB4499175	ThinClient v8.1	v8.0-ELR	2019-05 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems pclearstalecache_7f0c5cff499688bc34fdf52d5a6af3d97881d801.exe windows6.1-kb4499175-x86_6f1319c32d5bc4caf2058ae8ff40789ab10bf41b.msu
KB4499164	ThinClient v8.1	v8.0-ELR	2019-05 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
<b>nCare MX v10.1.2</b>			
KB4483229	nCareMX		2018-12 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4480979	nCareMX		2019-01 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4480961	nCareMX		2019-01 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4480961	nCareMX		2019-01 Delta Update for Windows 10 Version 1607 for x64-based Systems
KB4091664	nCareMX		2018-10 Update for Windows 10 Version 1607 for x64-based Systems
KB4480977	nCareMX		2019-01 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4346087	nCareMX		2019-01 Update for Windows 10 Version 1607 for x64-based Systems

KB4485447	nCareMX	2019-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4487038	nCareMX	2019-02 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4487026	nCareMX	2019-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4487026	nCareMX	2019-02 Delta Update for Windows 10 Version 1607 for x64-based Systems
KB4487006	nCareMX	2019-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4489907	nCareMX	2019-03 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4489882	nCareMX	2019-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4489882	nCareMX	2019-03 Delta Update for Windows 10 Version 1607 for x64-based Systems
KB4489889	nCareMX	2019-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4493478	nCareMX	2019-04 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4493470	nCareMX	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4493470	nCareMX	2019-04 Delta Update for Windows 10 Version 1607 for x64-based Systems
KB4346087	nCareMX	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
KB4499418	nCareMX	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4493473	nCareMX	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4495610	nCareMX	2019-05 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4498947	nCareMX	2019-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4494440	nCareMX	2019-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4497932	nCareMX	2019-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4494175	nCareMX	2019-05 Update for Windows 10 Version 1607 for x64-based Systems
KB4346087	nCareMX	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
<b>MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)</b>		
KB4499177	MedPresence-All-In-One	2019-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4346087	MedPresence-All-In-One	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
KB4497932	MedPresence-All-In-One	2019-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4494440	MedPresence-All-In-One	2019-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4498947	MedPresence-All-In-One	2019-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB890830	MedPresence-All-In-One	Windows Malicious Software Removal Tool x64 - May 2019
KB4493470	MedPresence-All-In-One	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4493478	MedPresence-All-In-One	2019-04 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB890830	MedPresence-All-In-One	Windows Malicious Software Removal Tool x64 - April 2019



KB4091664	MedPresence-All-In-One	2018-10 Update for Windows 10 Version 1607 for x64-based Systems
KB4489882	MedPresence-All-In-One	2019-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4489907	MedPresence-All-In-One	2019-03 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB890830	MedPresence-All-In-One	Windows Malicious Software Removal Tool x64 - March 2019
KB4487026	MedPresence-All-In-One	2019-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4487038	MedPresence-All-In-One	2019-02 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4485447	MedPresence-All-In-One	2019-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB890830	MedPresence-All-In-One	Windows Malicious Software Removal Tool x64 - February 2019
<b>ES4K v10.1.2 - BlizCon</b>		
KB4477029	BlizCon	2018-11 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4493478	BlizCon	2019-04 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4346087	BlizCon	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
KB4498947	BlizCon	2019-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4493470	BlizCon	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4497932	BlizCon	2019-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4494440	BlizCon	2019-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.1.2 - nCareRX2</b>		
KB4493478	nCareRX2	2019-04 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4346087	nCareRX2	2019-02 Update for Windows 10 Version 1607 for x64-based Systems
KB4498947	nCareRX2	2019-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4493470	nCareRX2	2019-04 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>ES4K v10.0.x - LSSB2</b>		
KB4494440	LSSB2	2019-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4497932	LSSB2	2019-05 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4498947	LSSB2	2019-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>PHP Updates</b>		
5.6.40 and 7.1.29		

For further questions concerning this notification please call Technical Support at (866) 376-6800.