

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®/nStream®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite an EasySuite 4K, nCare GX/MX/RX, nStream G3, EasyLink Router, Thin Client, LiveStream StatusBoard Engine, MedPresence® Stand-Alone, All-In-One Client, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites, nStreams, nCares, and Thin Clients connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: October 9, 2019

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

October 2019 Updates		
KB Number	Platform	Title
nStream v5.4 Updates		
KB4519976	nStream	2019-10 Security Monthly Quality Rollup for Windows 7 for x86-based Systems
KB890830	nStream	Windows Malicious Software Removal Tool - October 2019
VaultStream v5.3, 8.0.x, v9.0.x Updates – Windows® Server 2008		
KB4524157	VaultStream 2008	2019-10 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB4519976	VaultStream 2008	2019-10 Security Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - October 2019
KB4524102	VaultStream 2008	2019-10 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
VaultStream v10.x Updates – Windows® Server 2012		
KB4524156	VaultStream 2012	2019-10 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4520005	VaultStream 2012	2019-10 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - October 2019
KB4524104	VaultStream 2012	2019-10 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4521864	VaultStream 2012	2019-10 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
VaultStream v10.x Updates – Windows® Server 2016		
KB4519998	VaultStream 2016	2019-10 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4521858	VaultStream 2016	2019-10 Servicing Stack Update for Windows Server 2016 for x64-based Systems

KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - October 2019
KB4521858	VaultStream 2016	2019-10 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4524152	VaultStream 2016	2019-10 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4516061	VaultStream 2016	2019-09 Cumulative Update for Windows Server 2016 for x64-based Systems
EasyLink Router v8.0 Updates		
None		
nCare RX v8.0 and v8.1, nCare MX/ nStream GX v8.0, v9.0, and StatusBoard Engine, 8.0 and v8.1 Updates		
KB4516551	nCare RX/MX-SBE	2019-09 Preview of Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
		windows6.1-kb4507004-x64_a7d988d65422c8cfbbed141c95aeea37a8743167.msu
KB4516655	nCare RX/MX-SBE	2019-09 Servicing Stack Update for Windows Embedded Standard 7 for x64-based Systems
KB4516033	nCare RX/MX-SBE	2019-09 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4524102	nCare RX/MX-SBE	2019-10 Security and Quality Rollup for .NET Framework 3.5.1, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 7 and Server 2008 R2 for x64
		windows6.1-kb4507004-x64_a7d988d65422c8cfbbed141c95aeea37a8743167.msu
		windows6.1-kb4019990-x64_35cc310e81ef23439ba0ec1f11d7b71dd34adfe5.msu
KB4519976	nCare RX-SBE	2019-10 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x64-based Systems
KB4520003	nCare RX/MX-SBE	2019-10 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
KB4512486	nCare RX/MX-SBE	2019-08 Security Only Quality Update for Windows Embedded Standard 7 for x64-based Systems
HP Thin Client v8.0 and EasyLink Router v8.1 Updates		
KB4516655	Thin Client v8.0-ELR v8.1	2019-09 Servicing Stack Update for Windows Embedded Standard 7 for x86-based Systems
KB4516033	Thin Client v8.0-ELR v8.1	2019-09 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4524157	Thin Client v8.0	2019-10 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
KB4519976	Thin Client v8.0-ELR v8.1	2019-10 Security Monthly Quality Rollup for Windows Embedded Standard 7 for x86-based Systems
KB4520003	Thin Client v8.0-ELR v8.1	2019-10 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
KB4512486	Thin Client v8.0-ELR v8.1	2019-08 Security Only Quality Update for Windows Embedded Standard 7 for x86-based Systems
nCare MX v10.2.1		
KB4524152	nCare MX	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4519998	nCare MX	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

KB4521858	nCare MX	2019-10 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4512574	nCare MX	2019-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4516044	nCare MX	2019-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4516115	nCare MX	2019-09 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
MedPresence Stand-Alone v9.0.x and All-In-One Client (works with v9.0.x and 10.x)		
KB4521858	MedPresence Stand-Alone	2019-10 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4519998	MedPresence Stand-Alone	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4519562	All-In-One Client	2019-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB890830	All-In-One Client	Windows Malicious Software Removal Tool x64 - November 2019
KB4525236	All-In-One Client	2019-11 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4520724	All-In-One Client	2019-11 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4524095	All-In-One Client	2019-10 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.2.1 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4524152	EasySuite 4K Controller	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4519998	EasySuite 4K Controller	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4521858	EasySuite 4K Controller	2019-10 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.2.1 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4512574	EasySuite 4K Video Manager	2019-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
B4516115	EasySuite 4K Video Manager	2019-09 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4524152	EasySuite 4K Video Manager	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
EasySuite 4K v10.0.x – LiveStream StatusBoard 2		
KB4521858	LSSB2	2019-10 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4519998	LSSB2	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4524152	LSSB2	2019-10 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.32		

For further questions concerning this notice, please call Technical Support at (866) 376-6800



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas