

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: February 12, 2020

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

February 2020 Updates		
KB Number	Platform	Title
VaultStream v5.3, 8.0.x, v9.0.x Updates – Windows® Server 2008		
KB890830	VaultStream 2008	Windows Malicious Software Removal Tool x64 - February 2020
KB4539601	VaultStream 2008	2020-01 Preview of Monthly Quality Rollup for Windows Server 2008 R2 for x64-based Systems
KB2884256	VaultStream 2008	Security Update for Windows Server 2008 R2 x64 Edition
VaultStream v10.x Updates – Windows® Server 2012		
KB4502496	VaultStream 2012	2020-02 Security Update for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - February 2020
KB4538124	VaultStream 2012	2020-02 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4537821	VaultStream 2012	2020-02 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4537759	VaultStream 2012	2020-02 Security Update for Adobe Flash Player for Windows Server 2012 R2 for x64-based Systems
VaultStream v10.x Updates – Windows® Server 2016		
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - February 2020
KB4537764	VaultStream 2016	2020-02 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4524244	VaultStream 2016	Security Update for Windows Server 2016 for x64-based Systems
KB4534126	VaultStream 2016	2020-02 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
nCare MX v10.2.2		
KB4537759	nCare MX	2020-02 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4537764	nCare MX	2020-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4534126	nCare MX	2020-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64

EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4537759	EasySuite 4K Controller	2020-02 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4537764	EasySuite 4K Controller	2020-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4534126	EasySuite 4K Controller	2020-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4537759	nCare RX2	2020-02 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4537764	nCare RX2	2020-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4534126	nCare RX2	2020-02 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB4537759	LiveStream StatusBoard 2	2020-02 Security Update for Adobe Flash Player for Windows 10 Version 1607 for x64-based Systems
KB4537764	LiveStream StatusBoard 2	2020-02 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
PHP Updates		
5.6.40 and 7.1.33 (7.3.14 available upon request for v10.2.2 VaultStream only)		

For further questions concerning this notice, please call Technical Support at (866) 376-6800



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas