

Microsoft® Update and PHP Update Monthly Notification

Summary: This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

Products: EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

Date: July 15, 2020

Critical Security Bulletins (Obtained from Microsoft):

NOTE: Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

Approved Updates:

July 2020 Updates		
KB Number	Platform	Title
VaultStream v10.x Updates – Windows® Server 2012		
KB4566519	VaultStream 2012	2020-07 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4565541	VaultStream 2012	2020-07 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4566425	VaultStream 2012	2020-07 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB2978122	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3146723	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3037576	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3027209	VaultStream 2012	Update for Windows Server 2012 R2
KB3081320	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3012235	VaultStream 2012	Update for Windows Server 2012 R2
KB2989930	VaultStream 2012	Update for Windows Server 2012 R2
KB3127222	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64
KB3074545	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB4561666	VaultStream 2012	2020-06 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB3121261	VaultStream 2012	Update for Windows Server 2012 R2
KB2966828	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3044374	VaultStream 2012	Update for Windows Server 2012 R2
KB3054256	VaultStream 2012	Update for Windows Server 2012 R2
KB3126593	VaultStream 2012	Security Update for Windows Server 2012 R2

KB2973114	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3000483	VaultStream 2012	Security Update for Windows Server 2012 R2
KB4566519	VaultStream 2012	2020-07 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB4565541	VaultStream 2012	2020-07 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4566425	VaultStream 2012	2020-07 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB2894852	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3013410	VaultStream 2012	Update for Windows Server 2012 R2
KB3122651	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64
KB3134179	VaultStream 2012	Update for Windows Server 2012 R2
KB3185319	VaultStream 2012	Cumulative Security Update for Internet Explorer 11 for Windows Server 2012 R2
KB2966826	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3077715	VaultStream 2012	Update for Windows Server 2012 R2
KB3147071	VaultStream 2012	Update for Windows Server 2012 R2
KB2967917	VaultStream 2012	Update for Windows Server 2012 R2
KB3149157	VaultStream 2012	Update for Windows Server 2012 R2
KB3045999	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3132080	VaultStream 2012	Update for Windows Server 2012 R2
KB4033428	VaultStream 2012	Update for Windows Server 2012 R2
KB3012702	VaultStream 2012	Update for Windows Server 2012 R2
KB2919355	VaultStream 2012	Windows Server 2012 R2 Update
KB4087514	VaultStream 2012	Microsoft .NET Framework 4.8 Language Packs for Windows 8.1 and Server 2012 R2 for x64
KB2972103	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3097992	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3179574	VaultStream 2012	Update for Windows Server 2012 R2
KB3046737	VaultStream 2012	Update for Windows Server 2012 R2
KB3023219	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3115224	VaultStream 2012	Update for Windows Server 2012 R2
KB3088195	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3042058	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3123245	VaultStream 2012	Update for Windows Server 2012 R2
KB3175024	VaultStream 2012	Security Update for Windows Server 2012 R2
KB3071756	VaultStream 2012	Security Update for Windows Server 2012 R2
KB2972213	VaultStream 2012	Security Update for Microsoft .NET Framework 3.5 on Windows 8.1 and Windows Server 2012 R2 for x64-based Systems
KB3172614	VaultStream 2012	Update for Windows Server 2012 R2
KB3172614	VaultStream 2012	Update for Windows Server 2012 R2
KB3013769	VaultStream 2012	Update for Windows Server 2012 R2



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

VaultStream v10.x Updates – Windows® Server 2016		
KB4565511	VaultStream 2016	2020-07 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4565912	VaultStream 2016	2020-07 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4565628	VaultStream 2016	2020-07 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.82
KB4494175	VaultStream 2016	2020-01 Update for Windows Server 2016 for x64-based Systems
KB4103720	VaultStream 2016	2018-05 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4562561	VaultStream 2016	2020-06 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4561616	VaultStream 2016	2020-06 Cumulative Update for Windows Server 2016 for x64-based Systems
KB4052623	VaultStream 2016	Update for Windows Defender Antivirus antimalware platform - (Version 4.18.2001.10)
nCare MX v10.2.2		
KB4565511	nCare MX	2020-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4565912	nCare MX	2020-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4565628	nCare MX	2020-07 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 EasySuite 4K Controller (a.k.a. BlizCon)		
KB4565511	EasySuite 4K Controller	2020-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4565912	EasySuite 4K Controller	2020-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4565628	EasySuite 4K Controller	2020-07 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 EasySuite 4K Video Manager (a.k.a. nCare RX2)		
KB4565511	EasySuite 4K Video Manager	2020-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4565912	EasySuite 4K Video Manager	2020-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4565628	EasySuite 4K Video Manager	2020-07 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
EasySuite 4K v10.3 – LiveStream StatusBoard 2		
KB4565511	LiveStream StatusBoard 2	2020-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4565912	LiveStream StatusBoard 2	2020-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4565628	LiveStream StatusBoard 2	2020-07 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
MedPresence Cart v10.3		
KB4565912	MedPresence Cart	2020-07 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4565511	MedPresence Cart	2020-07 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4565628	MedPresence Cart	2020-07 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas

PHP Updates

5.6.40 and 7.1.33 (7.3.20 available upon request for v10.2.2 VaultStreams only)

For further questions concerning this notice, please call Technical Support at 1-800-848-9024



Image Stream Medical

One Monarch Drive Littleton, MA 01460 U.S.A.
www.olympusamerica.com/olympus-systems-integration

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas