

## Microsoft® Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the Image Stream Medical EasySuite®, LiveStream®, nCare®, and VaultStream® platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite and EasySuite 4K, nCare MX/RX, EasyLink Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v8.x and up VaultStream systems.

**Products:** EasySuites and nCares, connected to a network and VaultStream, as well as the LiveStream StatusBoard Engine.

**Date:** September 09, 2020

### **Critical Security Bulletins (Obtained from Microsoft):**

**NOTE:** Image Stream Medical **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

### **Approved Updates:**

September 2020 Updates		
KB Number	Platform	Title
<b>VaultStream v10.x Updates – Windows® Server 2012</b>		
KB4578013	VaultStream 2012	2020-08 Security Update for Windows Server 2012 R2 for x64-based Systems
KB4577066	VaultStream 2012	2020-09 Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB4576630	VaultStream 2012	2020-09 Security and Quality Rollup for .NET Framework 3.5, 4.5.2, 4.6, 4.6.1, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows 8.1 and Server 2012 R2 for x64
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.83
<b>VaultStream v10.x Updates – Windows® Server 2016</b>		
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.83
KB4576479	VaultStream 2016	2020-09 Cumulative Update for .NET Framework 4.8 for Windows Server 2016 for x64
KB4576750	VaultStream 2016	2020-09 Servicing Stack Update for Windows Server 2016 for x64-based Systems
KB4577015	VaultStream 2016	2020-09 Cumulative Update for Windows Server 2016 for x64-based Systems
<b>nCare MX v10.2.2</b>		
KB4577015	nCare MX	2020-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB4576479	nCare MX	2020-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4576750	nCare MX	2020-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>EasySuite 4K v10.3   EasySuite 4K Controller (a.k.a. BlizCon)</b>		
KB4576750	EasySuite 4K Controller	2020-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4576479	EasySuite 4K Controller	2020-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64

KB4577015	EasySuite 4K Controller	2020-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>EasySuite 4K v10.3   EasySuite 4K Video Manager (a.k.a. nCare RX2)</b>		
KB4576750	EasySuite 4K Video Manager	2020-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4576479	EasySuite 4K Video Manager	2020-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4577015	EasySuite 4K Video Manager	2020-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>EasySuite 4K v10.3 – LiveStream StatusBoard 2</b>		
KB4576750	LiveStream StatusBoard 2	2020-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB4576479	LiveStream StatusBoard 2	2020-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4577015	LiveStream StatusBoard 2	2020-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>MedPresence Tower v10.3</b>		
KB4577015	MedPresence Tower	2020-08 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB456479	MedPresence Tower	2020-08 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB4576750	MedPresence Tower	2020-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
<b>PHP Updates</b>		
5.6.40 and 7.1.33 (7.3.22 available upon request for v10.2.2 VaultStreams only)		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024



**Image Stream Medical**

One Monarch Drive Littleton, MA 01460 U.S.A.  
[www.olympusamerica.com/olympus-systems-integration](http://www.olympusamerica.com/olympus-systems-integration)

Image Stream Medical, Inc., is a wholly owned subsidiary of Olympus Corporation of the Americas