



## Microsoft Update and PHP Update Monthly Notification

**Summary:** This notification contains information about Microsoft security updates released this month and approved for implementation on the Gyrus ACMI, Inc., ES4K, LiveStream™, nCare™, and VaultStream™ platforms. The only updates listed are those considered to be of a security or critical nature. Only the updates listed below have been validated with the EasySuite™ HD and ES4K, nCare MX/RX, EasyLink™ Router, LiveStream StatusBoard Engine, and VaultStream systems.

Also included is information on the latest PHP updates that are approved for the v10.x and up VaultStream systems.

**Products:** EasySuite HD iOR systems, ES4K iOR systems, and nCare recorders, connected to a network and VaultStream server software, as well as the LiveStream StatusBoard Engine.

**Date:** September 12, 2023

### **Critical Security Bulletins (Obtained from Microsoft):**

**NOTE:** Gyrus ACMI, Inc. **does not approve** MS updates **KB975560, KB975562, KB2631813 or KB2780091** and strongly suggests they be excluded from automatic update scenarios. Failure to exclude these updates will result in equipment performance issues.

### **Approved Updates:**

<b>September 2023 Updates</b>		
<b>KB Number</b>	<b>Platform</b>	<b>Title</b>
<b>VaultStream v10.x Updates – Windows Server 2012</b>		
KB5030329	VaultStream 2012	2023-09 Servicing Stack Update for Windows Server 2012 R2 for x64-based Systems
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.890.0)
KB5030269	VaultStream 2012	Security Monthly Quality Rollup for Windows Server 2012 R2 for x64-based Systems
KB890830	VaultStream 2012	Windows Malicious Software Removal Tool x64 - v5.117
KB5030184	VaultStream 2012	2023-09 Security and Quality Rollup for .NET Framework 3.5, 4.6.2, 4.7, 4.7.1, 4.7.2, 4.8 for Windows Server 2012 R2 for x64
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.870.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.831.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.829.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.779.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.738.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.724.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.702.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.688.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.647.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.634.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.612.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.556.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.556.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.600.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.548.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.486.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.486.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.434.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.387.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.358.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.341.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.288.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.277.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.265.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.225.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.211.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.156.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.101.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.59.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.35.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1599.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1577.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1553.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1507.0)

KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1497.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1435.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1403.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1368.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1432.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1217.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1246.0)
KB2310138	VaultStream 2012	Security Intelligence Update for Microsoft Security Essentials - (Version 1.397.1296.0)
<b>VaultStream v10.x Updates – Windows Server 2016</b>		
KB5030213	VaultStream 2016	2023-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5030504	VaultStream 2016	2023-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5029924	VaultStream 2016	2023-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB890830	VaultStream 2016	Windows Malicious Software Removal Tool x64 - v5.117
<b>VaultStream v10.4 Updates – Windows Server 2019</b>		
KB5030178	VaultStream 2019	2023-09 Cumulative Update for .NET Framework 3.5, 4.7.2, 4.8 for Windows Server 2019 for x64
KB5030214	VaultStream 2019	2023-09 Cumulative Update for Windows Server 2019 (1809) for x64-based Systems
<b>nCare MX v10.2.3</b>		
KB5029924	nCare MX	2023-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5030504	nCare MX	2023-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5030213	nCare MX	2023-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	nCare MX	Windows Malicious Software Removal Tool x64 - v5.117
<b>nCare MX v10.4</b>		
KB5030213	nCare MX	2023-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB5030504	nCare MX	2023-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5029924	nCare MX	2023-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB890830	nCare MX	Windows Malicious Software Removal Tool x64 - v5.117
<b>ES4K v10.4 ES4K Controller (a.k.a. BlizCon)</b>		
KB5029924	ES4K Controller	2023-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64

KB5030504	ES4K Controller	2023-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5030213	ES4K Controller	2023-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	ES4K Controller	Windows Malicious Software Removal Tool x64 - v5.117
<b>ES4K v10.4   ES4K Video Manager (a.k.a. nCare RX2)</b>		
KB5029924	ES4K Video Manager	2023-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5030504	ES4K Video Manager	2023-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5030213	ES4K Video Manager	2023-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
KB890830	ES4K Video Manager	Windows Malicious Software Removal Tool x64 - v5.117
<b>ES4K v10.4 LiveStream StatusBoard 2</b>		
KB5030178	LiveStream StatusBoard 2	2023-09 Cumulative Update for .NET 3.5, 4.7.2, and 4.8 for Windows 10 Version 1809 for x64
KB5030214	LiveStream StatusBoard 2	2023-09 Cumulative Update for Windows 10 Version 1809 for x64-based systems
KB5029647	LiveStream StatusBoard 2	2023-09 Cumulative Update for .NET 3.5, 4.7.2, and 4.8 for Windows 10 Version 1809 for x64
<b>MedPresence Tower v10.4</b>		
KB5029924	MedPresence Tower	2023-09 Cumulative Update for .NET Framework 4.8 for Windows 10 Version 1607 for x64
KB5030504	MedPresence Tower	2023-09 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems
KB5030213	MedPresence Tower	2023-09 Cumulative Update for Windows 10 Version 1607 for x64-based Systems
<b>MAJ-2533 (UCES-4)</b>		
KB5030178	MAJ-2533	2023-09 Cumulative Update for .NET 3.5, 4.7.2, and 4.8 for Windows 10 Version 1809 for x64
KB5030214	MAJ-2533	2023-09 Cumulative Update for Windows 10 Version 1809 for x64-based systems
<b>CV Connectivity with KE v3.1 (VaultStream with Interfaces License)</b>		
KB5029244	KE v3.1	2023-08 Cumulative Update for Windows 10 Version 22H2 for x64-based Systems
KB5029649	KE v3.1	2023-08 Cumulative Update for .NET Framework 3.5, 4.8 and 4.8.1 for Windows 10 Version 22H2 for x64
KB890830	KE v3.1	Windows Malicious Software Removal Tool x64 - v5.116
<b>PHP Updates</b>		
V10.2.2 was Tested with PHP version 7.3.33 V10.4 was Tested with PHP version 7.4.30 7.3.33 is available upon request for v10.2.2 VaultStreams only. 7.4.33 is available upon request for v10.4 VaultStreams only.		

For further questions concerning this notice, please call Technical Support at 1-800-848-9024



**Gyrus ACMI, Inc.**  
 One 9600 Louisiana Ave. North, Brooklyn Park, MN 55445 U.S.A.  
[www.olympusamerica.com/olympus-systems-integration](http://www.olympusamerica.com/olympus-systems-integration)